

June 2020 Application update - BrailleNote Touch Plus Release notes

One-Handed Mode

- The BrailleNote Touch Plus now features a one-handed mode for Braille input, useful for users with dexterity impairments. The one-handed mode will only work with the HumanWare braille keyboard; the default two-handed mode will be active when using TouchBraille or a standard keyboard.
- To turn on one-handed mode, hold down **ENTER**, followed by the **Action button** (the button next to the headphone jack on the right edge of your device). Keep holding down **ENTER** and **the Action button** for 5 seconds until you hear "One-handed mode activated". Note that you **MUST** hold down **ENTER** before the Action button to activate the one-handed mode. This mode stays in effect until it is deliberately turned off.
- To return to two-handed mode, hold down the **ENTER** key followed by the Action Button until KeySoft announces "One-handed mode deactivated".

Full Scientific Calculator in KeyCalc

- KeyCalc now includes full scientific calculator functions
- Supports UEB and Nemeth input in addition to Computer Braille, LaTeX and French Math
- Switch between decimal and fraction results by pressing **Enter with G**
- Includes UEB and Nemeth symbol selector
- Access history of calculations and results by pressing **Left Thumb Key**
- Visual representation of calculations

New English Braille Learning tables:

- **Learn English braille in stages from 8 courses to choose from**
 - The following English learning tables are included:
 - NLS I.M.B.T (UEB 2015)
 - TSBVI Clusters
 - Braille in Easy Steps
 - Fingerprint (UEB)
 - Take Off Series
 - RNZFB STAR
 - S.A Syllabus Level
 - Mangold Method
 - The braille you read correlates with your current braille level.
- **FREE access to the full Take-Off UEB series curriculum in collaboration with RNIB**

- Teachers and student documents are available on HumanWare's support page.

EasyReader +

- **NEW: Added newspaper library RNIB Reading Services, UK**
- **NEW: Blind Low Vision, New Zealand**
A new library has been added called "Blind Low Vision" from New Zealand.

Fixes and Enhancements:

- Opening TXT files in KeyWord is more efficient with speed improvements.
- Increased limit for TXT warning from 300KB to 10MB.
- When Fixed issues working with multiple documents in KeyWord.
- Fixed fast switching between apps when using a custom Main Menu.
- Mapped the action button to record an audio message when using What's app.
- Fixed issues with Arabic documents that include Tashkeels in KeyWord.
- Various other improvements.