explorē 12

User Guide

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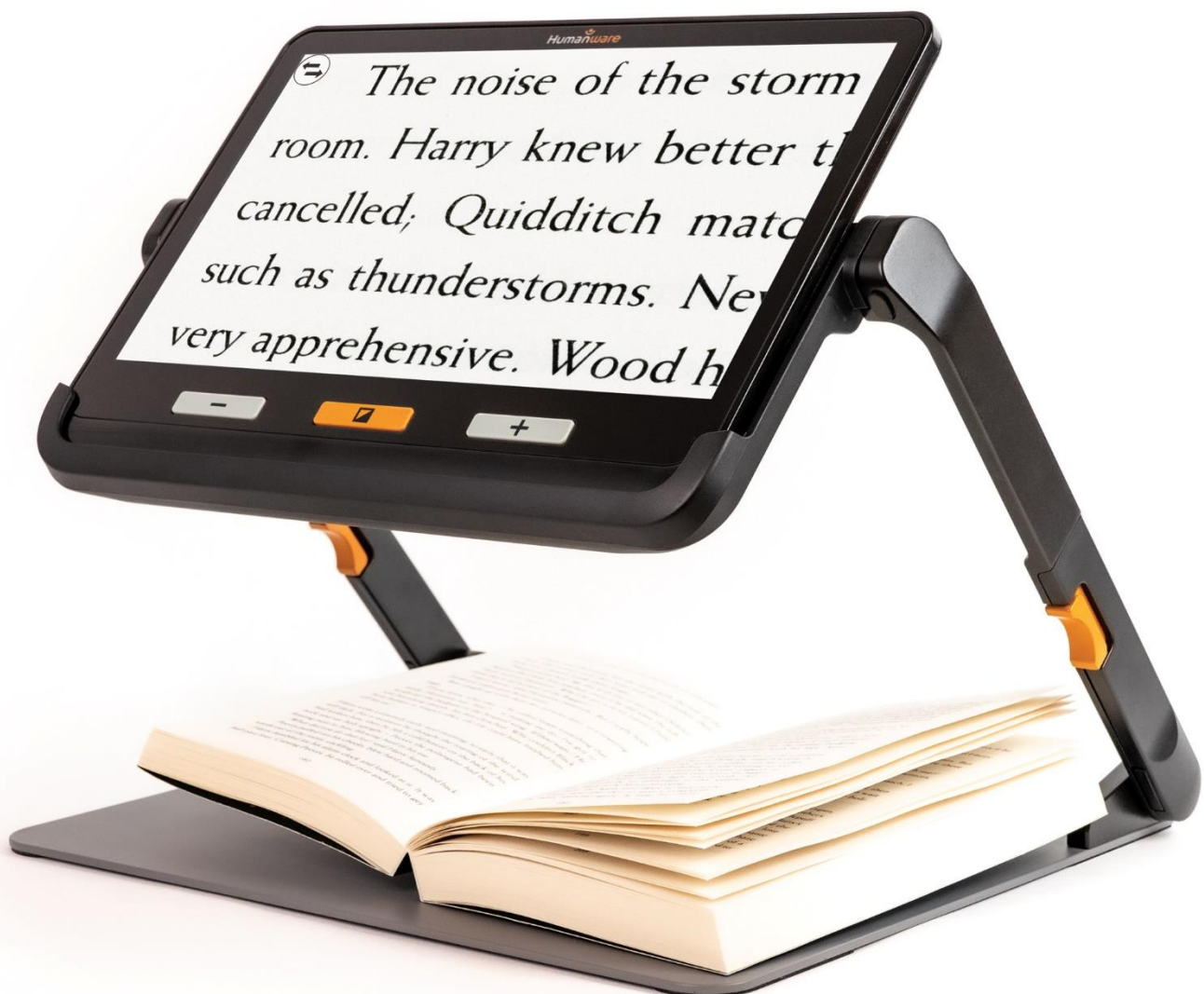
TABLE OF CONTENTS

Chapter 1 – Overview	5
Introduction	5
Box Contents.....	5
Physical Description.....	6
Getting Started	8
Chapter 2 – Using Your System	10
Powering On	10
Powering Off	10
Magnifier Mode	10
Zooming In and Out	11
Horizontal panning of a live image (folding stand only)	11
Switching Cameras	12
Changing Contrast	12
Turning off and on the lights	12
Lines and Blinds.....	13
Autofocus	13
Freezing an Image	14
Navigating on a Frozen Image.....	14
Gallery Mode.....	15
Accessing the Gallery	15
Deleting Images from Gallery	15
Using an External Screen	16
Chapter 3 – Exporting Images to Computer	16
Chapter 4 – Menu.....	17
Accessing the Menu and Changing Settings	17
Chapter 5 – Updating Your System.....	19
Chapter 6 – Recharging Your System.....	19
Chapter 7 – Troubleshooting	21
Appendix A – Safety and Maintenance	23
Appendix B – Specs	24
Appendix C – FCC Info.....	25
Appendix D – Warranty.....	26

CHAPTER 1 – OVERVIEW

INTRODUCTION

Thank you for purchasing the HumanWare explorē 12, the simple to use portable electronic handheld magnifier with a large, easy-to-read 12" screen. With your explorē 12, you'll be reading those fine print materials in a matter of seconds.

Being magnifier users ourselves, we at HumanWare know that in addition to the best available image quality, a good magnifier needs to be simple, portable, and comfortable to use and be ready to use immediately when and where you need it. Your explorē 12 was designed with just that in mind, for which we take considerable pride in offering you a quality product that will meet both your needs and expectations. Carefully read the operating and safety instructions described in this user guide before using your device. If you have any questions about your system, please contact HumanWare Customer Service by consulting the contact information in the back of this user guide.

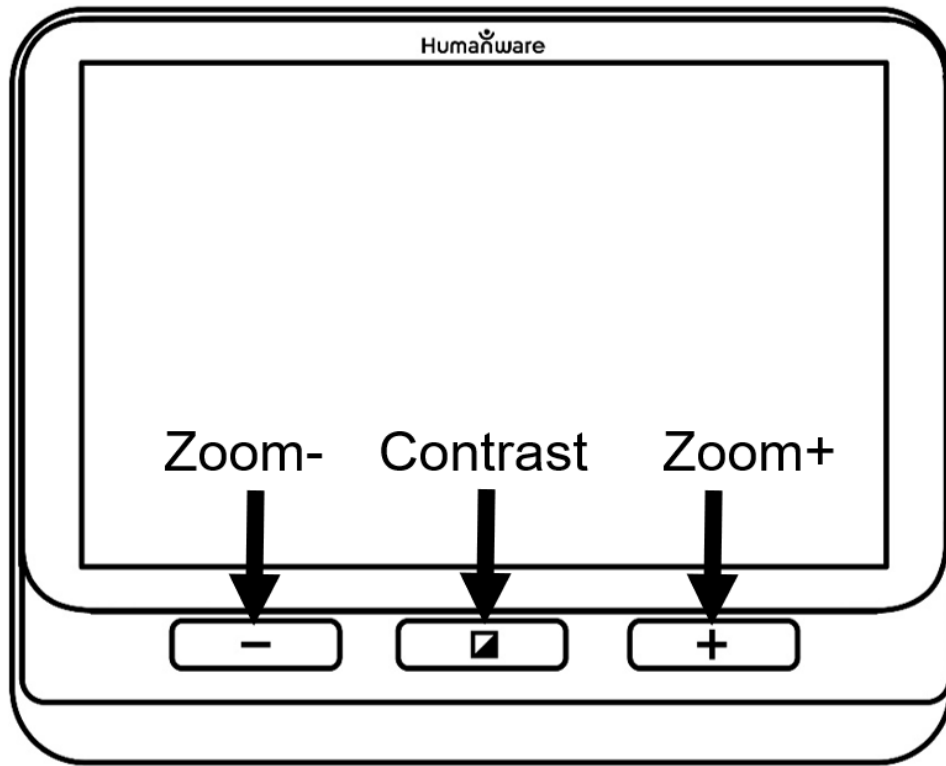
Copyright 2021 by Technologies HumanWare Inc., all rights reserved. No part of this publication may be reproduced or distributed in any form or by any means or stored in a database or retrieval system without the prior written permission of Technologies HumanWare Inc.

BOX CONTENTS


- explorē12 electronic magnifier
- USB charger/cable
- explorē12 reading stand (if purchased)
- High-Resolution video cable
- LCD Protector
- Microfiber cloth
- Protective case
- User Guide

PHYSICAL DESCRIPTION


Front face


Left side

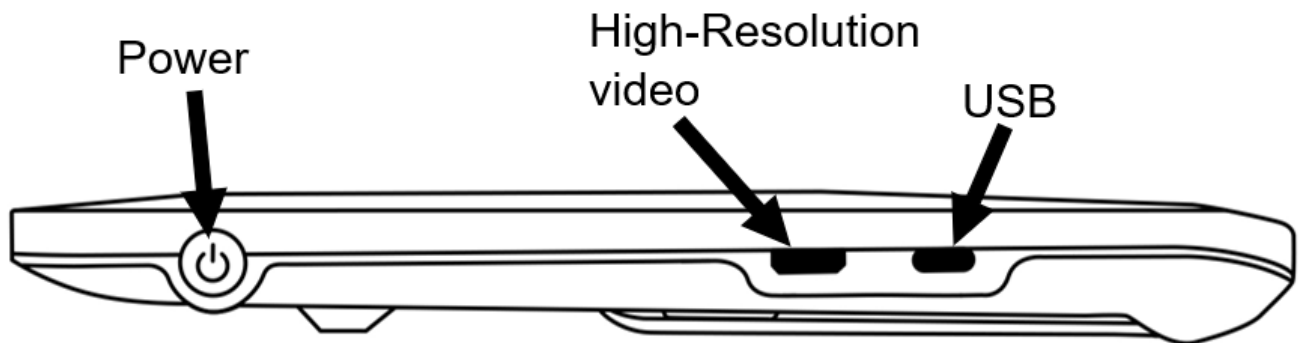Top edge

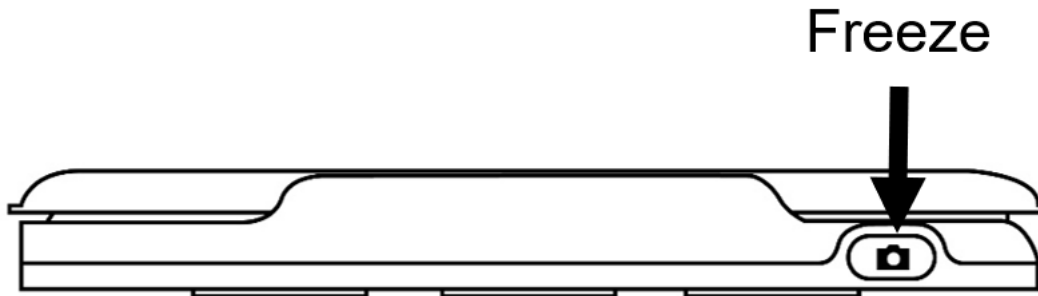Backside

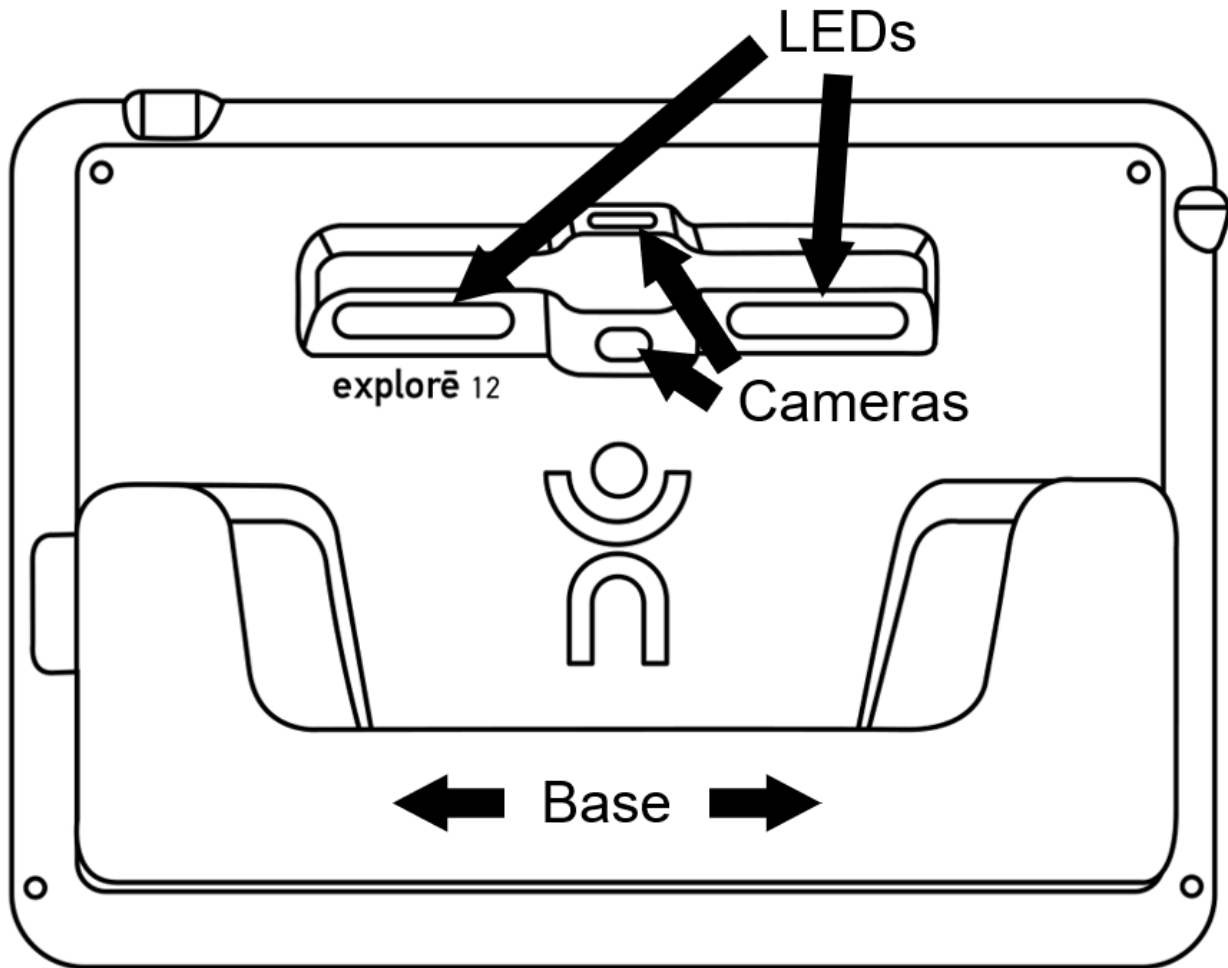
GETTING STARTED



1. Connect the USB cable to the USB charger.
2. Hold explorē 12 so that the screen side is facing you. Then, carefully connect the cable to the USB port located on the left side of your device and plug the charger into an outlet.



IMPORTANT: Use caution while inserting the cable into the USB port.

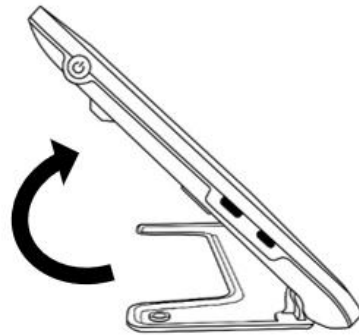3. Let your device recharge until your battery is full. The device will become warmer during the charge – this is normal.
4. Once the device is fully charged, carefully disconnect the cable.
5. Press **Power** or unfold the reading base to start using the device.

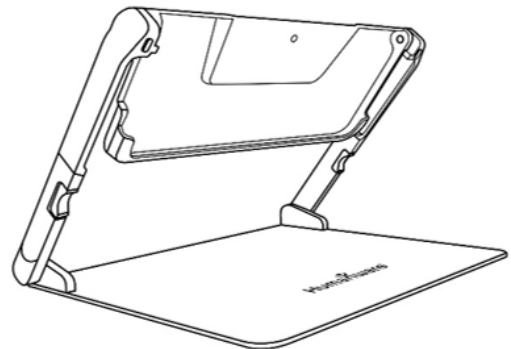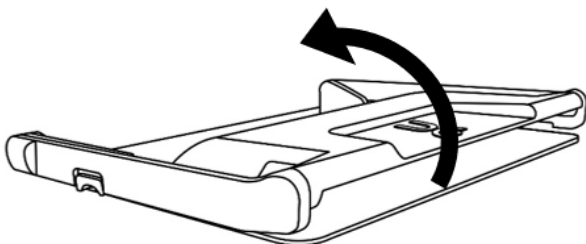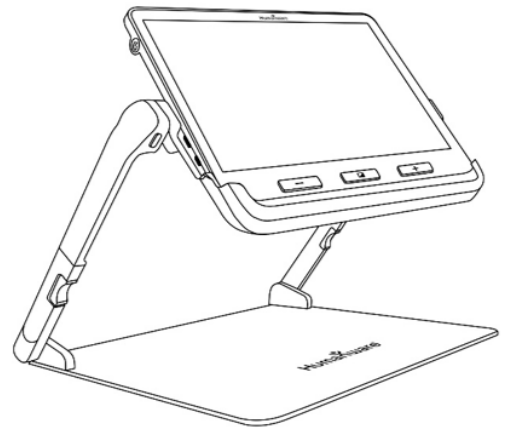
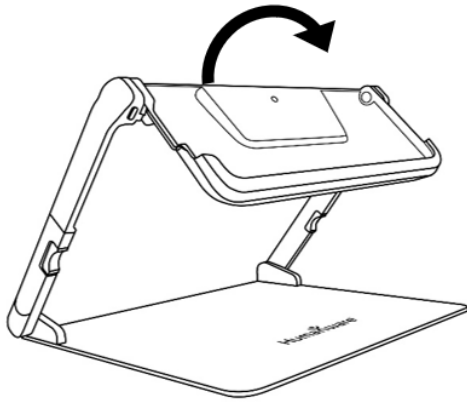
6. Alternatively, you can unfold the reading stand (if purchased) and dock the device on the tray to use it in "Stand Mode."

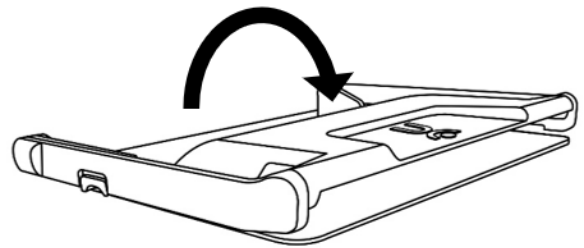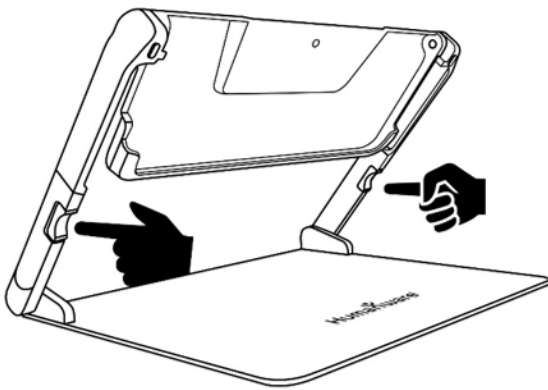7. When you are done working in "Stand Mode," remove the device from the tray, then press the two orange buttons on each side of the stand to fold it back.


CHAPTER 2 – USING YOUR SYSTEM

POWERING ON

To start your magnifier, simply press **Power** on the left side of the device. The device can then be held by the sides, just like you would hold a tablet.

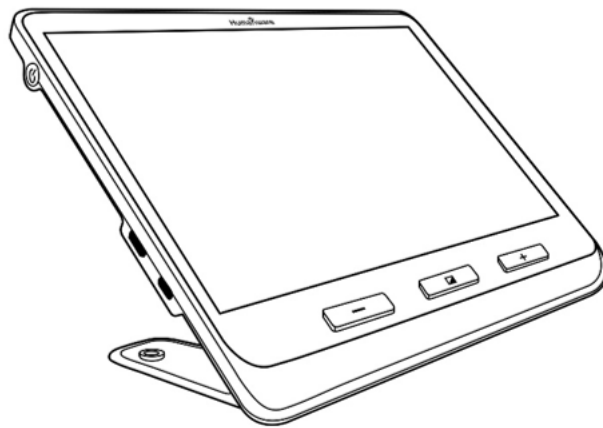Alternatively, you can a) turn on your device by unfolding the base behind the device or b) opening the reading stand and inserting the device, then use the device with what you wish to read.

After a few seconds, the explorē 12 screen will turn on and display the image from the camera. This is the magnifier mode.

POWERING OFF

To turn off your device, simply press the **Power** button.

MAGNIFIER MODE

Magnifier mode is explorē 12's primary mode of operation. With the base closed, you will be able to magnify distant objects. With the base opened, you will be able to magnify close objects or documents on a table. With the device placed on the reading stand, you will be able to magnify close objects or documents under the device without physically moving them.

ZOOMING IN AND OUT

To zoom in, simply press or press and hold, **Zoom+** until the image is of the appropriate size. Similarly, you can zoom out by pressing, or pressing and holding, **Zoom-** until you are satisfied with the image size.

Alternatively, you can enable the **pinch and zoom** gesture (decrease and increase the distance between two fingers) from the menu to zoom in and out of a live image (see Ch. 4).

You can slide two fingers from top to bottom across the middle of the screen, and the image will zoom out to 1x, allowing you to reposition your material. The image will return to the previous zoom level once you release your fingers from the screen.

HORIZONTAL PANNING OF A LIVE IMAGE (FOLDING STAND ONLY)

While viewing a document or an object in magnifier mode using the foldable reading stand (optional purchase), you can pan left and right through the image by sliding your finger across the screen in the direction you wish to pan.

For zoom levels 4x and higher, you can also display, from the menu, left and right arrows at the bottom of the screen (see Ch. 4). Tapping those arrows will take you all the way to the left (or right) of the camera's viewable area.

Note that you must use the Near camera on the stand for the Horizontal panning feature to work.

SWITCHING CAMERAS

By default, the magnifier mode is set to the Near Camera view when the base or the stand is opened, and the device is in magnifier mode. However, it is possible to toggle to the Distance Camera by pressing the **Toggle Camera** button at the top right of the screen.

Alternatively, you can toggle the camera view by selecting **Toggle Camera** in the menu (see Ch. 4).

CHANGING CONTRAST

explorē 12 allows you to customize contrast to better suit your needs and preferences. Simply press **Contrast** to change contrast until you find one you are comfortable with.

The available default contrast modes are listed below:

- Full color, black on white, white on black, black on yellow, yellow on black

Other contrast modes are also available from the menu (see Ch. 4):

- Black on orange, orange on black, black on cyan, cyan on black, black on purple, purple on black, black on green, green on black, blue on yellow, yellow on blue, white on blue, blue on white

TURNING OFF AND ON THE LIGHTS

Press and hold **Zoom+** and **Zoom-** simultaneously to turn lights off and back on when necessary. Note that the Near camera must be selected to turn on the lights.

LINES AND BLINDS

With the explorē 12, you can add a horizontal line or blinds over your image for easier reading.

To add or remove a line, press and hold **Contrast** and **Zoom+** simultaneously.

To add or remove blinds, press and hold **Contrast** and **Zoom-** simultaneously.

AUTOFOCUS

explorē 12 features an autofocus that can be set to **locked**, **unlocked**, or **manual** from the menu (see Ch. 4).

To temporarily lock the autofocus, double tap on the screen while magnifying an image. A padlock icon will appear at the top right corner of the screen. Double-tap again to unlock the autofocus.

Note that changing the zoom level of the image also unlocks the temporary autofocus lock.

When autofocus is set to **unlocked** (by default), explorē 12 will dynamically adjust its focus to the object currently being viewed. The focus can be forced by touching anywhere on the screen.

When autofocus is set to **locked**, or when the magnifier is used in base mode, explorē 12 will maintain its current focus regardless of any movements or changes.

When autofocus is set to **manual**, you can touch any position on the screen to have explorē 12 focus on that specific location. Note that the **manual** setting only works in distance viewing mode and stand mode. Base mode will continue to be locked when **manual** mode is selected.

FREEZING AN IMAGE

explorē 12 enables you to freeze an image while in magnifier mode for easier reading, a handy feature when reading product labels on stores' top shelves. To freeze an image, press the **Freeze** button while aligning your magnifier towards what you wish to read. This will take a snapshot of what your device was oriented towards. The snapshot will be displayed on your screen until you decide to return to Magnifier mode.

To return to magnifier mode, simply press **Freeze** once again.

NAVIGATING ON A FROZEN IMAGE

To zoom in on still images, simply press or press and hold **Zoom+** until the image is of the appropriate size. Similarly, you can zoom out by pressing or pressing and holding **Zoom-** until you are satisfied with the image size.

Alternatively, you can enable the **pinch and zoom** gesture (decrease and increase the distance between two fingers) from the menu to zoom in and out of a frozen image (see Ch. 4).

Hold your finger down and slide it across the screen to see other parts of a frozen image.

As with a live image, you can press **Contrast** to change contrast until you find one that is comfortable.

GALLERY MODE

To save a live or frozen image to your Gallery, press and hold **Freeze** for a half-second while in Magnifier mode. You should see a Save message displayed briefly on your screen, meaning your image is now saved inside explorē 12's Gallery.

Alternatively, you can enable the **tap and hold** gesture (keep your finger down on the screen) from the menu to save an image in the Gallery (see Ch. 4).

ACCESSING THE GALLERY

To access your images in the Gallery, press **Contrast** and **Freeze** simultaneously. Then, swipe your finger left or right on the screen to select an image. You can also press the arrows on the screen to choose an image.

You can zoom in and out of saved images by pressing or pressing and holding **Zoom+** and **Zoom-**. Alternatively, you can enable the **pinch and zoom** gesture (decrease and increase the distance between two fingers) from the menu to zoom in and out of a saved image (see Ch. 4).

To exit the Gallery and return to Magnifier mode, press **Back** (on-screen) or press and hold **Contrast**.

DELETING IMAGES FROM GALLERY

To delete an image from the Gallery, access the Gallery and navigate to the image you wish to delete (by swiping your finger left or right on the screen or using the arrow buttons), then press and hold **Freeze**.

Alternatively, you can enable the **tap and hold** gesture (keep your finger down on the screen) from the menu to delete an image (see Ch. 4).

You will have the option to Delete or Cancel. Press Delete to confirm the deletion. To cancel deletion, press Cancel or **Contrast**.

You can delete all your images at once from the menu (see Ch. 4). Press and hold **Contrast** and select the item **Delete Gallery** that will appear on the screen.

USING AN EXTERNAL SCREEN

For a larger image, you may want explorē 12's content to be displayed on an external screen (like a TV). To do so, connect your explorē 12 to an external screen using the included High-Resolution video cable. Please refer to Appendix B for a list of supported High-Resolution video formats.

CHAPTER 3 – EXPORTING IMAGES TO COMPUTER

To export Gallery images saved on explorē 12 to your computer, connect your device with the USB cable provided. You should see a "PC connected" message appear on your device. You can now copy and paste the images from your Gallery to your computer using your computer's file browser.

When connected to a personal computer, the device will act exactly like a USB thumb drive or an external drive. Gallery images are in the Images folder on your explorē 12.

When copying or deleting images from the device, never disconnect the USB cable as some images could be lost in the process.

CHAPTER 4 – MENU

ACCESSING THE MENU AND CHANGING SETTINGS

Press and hold **Contrast** to enter the menu. You can navigate the menu by swiping up or down with your finger on the screen, pressing the up/down arrows, or pressing **Zoom+** to scroll up and **Zoom-** to scroll down.

To select an item, press on it. To go back to the previous menu, press **Contrast** or **Back**. Press **Contrast** again to return to Magnifier mode. To quickly exit the menu, press and hold **Contrast** or **Back**.

The menu contains the following items:

- **Gallery** (Access Gallery images)
- **Toggle camera:** Distant Camera*, Near Camera (Near Camera* in base or stand mode)
- **Autofocus:** Unlocked*, Locked (Locked* in base mode), Manual (Locked in base mode)
- **Screen Brightness** (Change the screen's brightness level): 1 to 9 (8*)
- **Touchscreen** (submenu, tap to enable gestures and vibrations)
 - **Pinch & zoom** (decrease/increase distance between two fingers to zoom out/in)
 - **Vibrations**
 - **Tap & hold** (hold a finger down in Magnifier to save an image to Gallery)
 - **Pan buttons** (for zoom 4x or higher on folding stand only; displays left and right arrows at the bottom of the screen to go

to the far left or right of the camera's viewable area while magnifying a live image)

- **Auto-off** (Choose how long the device stays on): 1 minute, 2 minutes, 5 minutes*, 10 minutes
- **Preferred Zoom** (Change the default zoom level when you open your device): Previous (the zoom level you were at when you closed your device), 2x* to 30x; 2x* to 15x (in Stand mode with Near camera)
- **Contrast** (submenu)
 - **Preferred Contrast** (select contrast in which your explorē 12 starts up): Previous* (last used contrast), Color, or choice of enabled contrasts
 - **Menu** (change menu colors): Black on white*, White on black, Black on yellow, Yellow on black, Black on orange, Orange on black, Black on cyan, Cyan on black, Black on purple, Purple on black, Black on green, Green on black, Blue on yellow, Yellow on blue, White on blue, Blue on white
 - **Enabled contrasts** (select enabled color pairs for magnifier mode): Black on white*, White on black, Black on yellow, Yellow on black, Black on orange, Orange on black, Black on cyan, Cyan on black, Black on purple, Purple on black, Black on green, Green on black, Blue on yellow, Yellow on blue, White on blue, Blue on white
- **Lines and blinds:** Line, Blinds, None*
- **Lights:** On*, Low, Off
- **Menu Text Size:** Small, Medium*, Large

- **Language:** English US*, English UK, French (CA), French (FR), Spanish (US), Spanish (ES), German, Dutch, Danish, Polish, Portuguese (PT), Portuguese (BR), Norwegian, Italian, Swedish, Finnish, Russian, Czech, Japanese, Korean, Icelandic, Estonian, Turkish, Lithuanian, Hungarian, Latvian, Arabic, Chinese (Simplified), Chinese (Traditional), Hebrew, Georgian, Farsi, Kurdish
- **Display:** 50Hz, 60Hz*
- **Delete Gallery:** Remove all images on your device.
- **Factory Defaults:** Clear all personal settings on the device to factory defaults.
- **About** (View information about your device): Set date and time, Software, Hardware, Serial Number.

CHAPTER 5 – UPDATING YOUR SYSTEM

First, go to: <http://support.humanware.com/en-canada/support/explore>

Then, download the latest version of the explorē 12 software. Next, connect your explorē 12 to your computer using the provided USB cable. Using your computer's file explorer, copy and paste or drag and drop your downloaded file directly into your explorē 12 root directory. When the transfer is completed, unplug your USB cable from your computer. Restart your explorē 12. Your device should automatically update itself.

CHAPTER 6 – RECHARGING YOUR SYSTEM

IMPORTANT: The battery becomes warm during recharging. This is normal. Do not recharge the battery near a heat source.

To recharge the battery using a household power outlet:

- Connect the USB cable to explorē 12's USB charger. Then, carefully connect the USB cable to your device and plug the charger into an outlet.
- Let your device charge until your battery is full. It should take approximately 3.5 hours to recharge an empty device.

The image below will show momentarily when the battery starts charging or when the device is turned on during the recharge process.

The lightning bolt icon means your device is being charged. The number to the right represents the percentage of battery charge remaining.

Due to lower voltage, charging from the computer USB may be slower. In some cases, your device may not charge at all, depending on the amount of power supplied by the computer. The most effective way to recharge the device is to use the provided USB cable connected to the USB charger and recharging from a household power outlet.

Although it may lengthen the charging time, the device can be used when it is recharging.

Please note that if the battery is fully discharged or hasn't been used for a very long period, there will be a delay of up to a few minutes before the device shows any activity. This is normal.

Also, note that your explorē 12 will become warm when in use or when being charged like all electronic devices – this is normal.

Battery safety precautions:

- Do not disassemble or modify the battery.

- Use only the specified HumanWare charger.

There is a risk of overheating, fire, or explosion if the battery is put in a fire, heated, subjected to impact, put in contact with water, or if its terminals are shorted.

CHAPTER 7 – TROUBLESHOOTING

explorē 12 does not power on:

Press and hold **Power**. If the unit still does not turn on, your device's battery may need recharging.

Connect the provided USB cable with the USB charger into a power outlet or just the USB cable into a running computer. If the battery was completely discharged, it might take a few minutes before the device shows any signs of activity (see Ch. 6 for more info).

Screen is black:

The image may be black if the unit is lying on a table or any flat surface. Lift your magnifier to see whether this is the reason why your screen is black.

If this does not solve the problem, try restarting your device.

Image is out of focus in magnifier:

Make sure Autofocus is set to Unlocked in the menu. Next, move the device back and forth over the document you are viewing. This will make explorē 12 re-focus on the document.

explorē 12 shuts down after inactivity:

This is a normal behavior intended to preserve the battery. By default, explorē 12 will shut down after 5 minutes of inactivity. However, it is

possible to configure the default auto-off timer to 1 minute, 2 minutes, 5 minutes, or 10 minutes.

Colors are difficult to detect:

First, make sure there is sufficient ambient light around you.

Choose a different color combination by pressing **Contrast** while in magnifier mode.

There is a lot of glare:

Tilt the object you are looking at in a different direction to reduce glare. You can also try turning off the lights by pressing and holding **Zoom+** and **Zoom-** simultaneously.

explorē 12 does not respond:

Make sure explorē 12 is not connected to power via USB cable; if it is, unplug it and then press **Power** for 10 seconds. The system will shut down. Then, press **Power** for one second to turn explorē 12 back on.

Other problems:

Restore to factory defaults in the menu.

Press and hold **Contrast**. Scroll down to Factory Defaults and select Yes.

APPENDIX A – SAFETY AND MAINTENANCE

You must not stare at the LED light source located on the backside of the magnifier.

Keep away from water or high humidity. Do not submerge.

Power off your explorē 12 before cleaning it.

Use only cables provided with your explorē 12.

Battery:

The explorē 12 has an internal rechargeable Li-Ion battery. A qualified HumanWare technician must replace the battery.

- Do not use or leave the unit near a heat source or fire. Do not store in high temperatures.
- Only use USB-compliant power supplies to charge the battery.
- Do not disassemble or modify the unit.
- Do not immerse the unit in water, do not allow it to get wet.
- Do not strike or throw the unit.
- Do not pierce or strike the unit with sharp objects or a hammer.
- If the battery leaks and its liquid gets into the eyes, do not rub the eyes. Instead, rinse the eyes with clean running water and seek immediate medical attention to prevent injury.
- If the battery leaks and its liquid comes into contact with skin, immediately rinse the area with clean running water to prevent injury.

Cleaning the LCD screen:

Make sure your device is off and unplugged.

Make sure you only use microfiber cloths to clean the LCD screen like the one included with your device.

Carefully and gently wipe the microfiber cloth across the LCD screen.

If you choose to clean your LCD screen with a liquid cleaner, make sure this cleaner is appropriate for LCD screens. Distilled water, isopropyl alcohol, and vinegar-water solutions (no more than 10% vinegar) are safe cleaning liquids for your LCD screen.

Use only a minimal amount of liquid to prevent it from getting inside the device. If liquid seeps in, damage to the device might occur. Make sure no liquids remain on your screen. Repeat the previous steps for spots that are harder to remove.

Never use household cleaners to clean your explorē 12.

Never spray any liquid directly on your LCD screen.

Never use cleaning products that contain ethyl alcohol, ethyl acid, ammonia, acetone, or methyl chloride.

Never use a paper towel to clean your LCD screen.

APPENDIX B – SPECS

Tablet size: 282.5 x 202 x 24 mm

Stand size: 345 x 270 x 35.5 mm

Display: 12" ISP LCD

Weight: 1134 g

Weight with stand: 2268 g

Power Input: USB Type C connector. 5V@3A, 9V@2A or 12V@1.5A

Battery life: 3.5 hours base mode and/or distance mode and 2 hours in stand mode with LEDs on

Charge Time: 4 hours (when the device is off)

Supported High-Resolution video formats:

1080p 60Hz, 1080p 50Hz, 720p 60Hz, 720p 50Hz

No support of DVI emulation

Operating conditions:

Temperature: 10 °C to 40 °C / 50 °F to 104 °F

Humidity: 20% to 80%

Transport and storage conditions:

Temperature: -20 °C to 60 °C / -4 °F to 140 °F

Humidity: 10% to 95%

APPENDIX C – FCC INFO

FCC Statement:

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

FCC Warning:

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television

reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Note: This unit was tested with shielded cables on the peripheral devices. Shielded cables must be used with the unit to ensure compliance.

APPENDIX D – WARRANTY

HumanWare warrants explorē 12, effective from the date of purchase, to be free of any defects in material and workmanship for a period of 2 years unless otherwise required by law in the country or region of purchase.

This warranty is non-transferable and applies to all cases where the damage is not a result of improper use, mistreatment, or negligence. Improper use is use of this device other than described in this manual. In no event shall HumanWare or its distributors be liable for indirect or consequential damages.

No replacement or repair covered by the warranty will be carried out unless the system is accompanied by a copy of the original bill of purchase. Please keep your original receipt. If the system has to be returned, please use the original packaging.

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